



## **Instructions for Nurses that will be applying medical grade IV Bag Stickers**

1. Identify the side of your IV bag that does not have the measurement line indicator.
2. Once you or your patient decides which design will be utilized, identify the appropriate sized sticker to match your IV bag size. (i.e. 1000 mL sticker for 1000 mL IV bag) To double check the size of the sticker you have identified, flip the sticker to the paper backing which will indicate the given size of the sticker.
3. Peel off the protective layering to ensure high quality and then dispose of this protective layering.
4. Peel only the top paper backing and adhere to the top of your IV bag. This adherence allows a nurse to lift the sticker throughout the care at their convenience to view any additional information that this side of the IV bag contains.
5. Inform your patient that more resources can be found through scanning the QR code.
6. Once care has concluded, dispose of the IV bag sticker with the IV bag.

Thank you for positively impacting your patients' journey!

Please contact us for further questions at:  
info@4umedicaldesigns.com or 609-605-4907