



Instructions for Nurses that will be applying medical grade IV Bag Stickers

1. After the Intravenous (IV) Therapy is initiated, identify the side of your IV bag that does not have the measurement line indicator.
2. Once you or your patient decides which design will be utilized, identify the appropriate sized roll to match your IV bag size. (i.e. 1000 mL sticker roll for 1000 mL IV bag) To double check the size of the sticker you have identified, flip the sticker to the paper backing which will indicate the given size of the sticker.
3. Using the perforation in between each sticker on the roll, tear off the selected sticker from the roll.
4. Peel off the protective layering to ensure high quality and then dispose of this protective layering.
5. Identify the blank text bubble of your IV bag sticker. The nurse has the option to marker in any information (i.e. medication or patient information) that they would like visible as a reminder.
6. If the side of the IV bag that does not have a measurement line indicator contains any information, please peel only the top partial adhering option and adhere to the top of your IV bag. This option allows for a nurse to lift the sticker to view any additional information that this side of the IV bag contains.
7. If the side of the IV bag that does not have a measurement line indicator is completely blank and the nurse prefers to adhere the entire sticker to the IV bag, please peel both the partial and full adhering options and adhere to your IV bag.
8. Inform your patient that more resources can be found through scanning the QR code.
9. Once intravenous therapy has concluded, dispose of the IV bag sticker with the IV bag.

Thank you for positively impacting your patients' journey!